CompoMaster

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

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nl

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PRODUCT DESCRIPTION

CompoMaster are impregnated with diamond grains in a synthetic rubber binder. CompoMaster Coarse are intended for finishing and polishing of composite restorations and CompoMaster is for super-polishing them. For contouring, Dura-Green Stones or Dura-Green Stones Fine is recommended. For further super-polishing, DirectDia paste or SuperBuff polishing discs is recommended.

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds. ●Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points. ●Mount an unmounted wheel onto the mandrel securely. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Always use an unmounted wheel in clockwise motion. ●Use local dust extractor and protective equipment while using. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use. ●Dispose in accordance with relevant local and national regulation.

STORAGE AND STERILIZATION

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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